

## INSTRUCTION SHEET FOR INSPECTING DIASTAT<sup>®</sup> ACUDIAL<sup>™</sup>

Please follow these 4-step instructions to carefully inspect the Diastat<sup>®</sup> AcuDial<sup>™</sup> for any crack. Be careful not to fully remove the cap from the syringe during inspection.



Step 1: Holding syringe in one hand with large end of oval pointing up and down. Place index finger in contact with cap to apply counter force when cap is opened.



Step 2: Using the opposite hand grasp the cap firmly with your index finger, applying pressure to the seal pin to hold it in place.



Step 3: Apply downward force with your thumb to create an opening 2/8 to 3/8" to view base of syringe tip and inspect for crack. Adjust angle to give best view of syringe tip.



**Example of a Crack at the Base of the Syringe Tip**



Step 4: While maintaining pressure on the seal pin apply an upward force to close the cap. Rotate the syringe 180 degrees and repeat steps 3 and 4.

